



Pressalit Care

Flexible Bathroom Solutions
and Adult Changing Tables



The three areas of the bathroom

The key to a functional bathroom is a systematic evaluation of the room's potential taking into consideration the end user and caregiver's needs. An evaluation needs to be made of space requirements, turning areas, and the typical turning radius at the washing, toileting and showering areas of the bathroom.

WASHING Having the correct height to the sink can be critical to many people. The sink bracket is ideal if many people share the same sink. Users who need to sit or stand have the best opportunity to be independent - and thus achieve greater dignity and satisfaction - when the sink can be adjusted to the correct height at all times.

THE TOILET AREA Using fold-down grab bars which are both vertically as well as horizontally adjustable enables different users to find the proper support when using the toilet. The length and stability of the fold-down grab bars provide security during transfers to and from the toilet.

THE SHOWER AREA A high degree of flexibility is the key when designing a functional bathroom. Using a shower seat which is both vertically and horizontally adjustable means that good working height can be obtained for the caregiver while the user sits in the chair. Adding a hand and shower rail corner with a shower faucet bracket on a wall track means that both right as well as lefthanded users' needs can be accommodated. The chair can be used to create space for user as well as caregiver.

Contact us, if you have questions or need inspiration and guidance in your process of planning your next functional bathroom.





■ A world of color



The flexibility of the PLUS system is also reflected in the wide choice of colors available. Thanks to the variety of both subtle and intense colors, it is easy to choose exactly the right color combinations to suit the bathroom's main purpose while satisfying user needs for an aesthetic and plainly laid-out design.

Colors can help to add a uniquely personal and individual feel while also assisting the user to find their way around in relation to the bathroom's various functions. As in traffic, colors in the bathroom can provide direction and function as signals.

Using color contrasts provides an important and invaluable help with daily routines, especially for many older and visually impaired people.

Wall tracks
- Standard colors



182 PALE GREY



112 ANTHRACITE

Fold-down grab bars and shower seats
- Standard colors



000 WHITE



112 ANTHRACITE

Fold-down grab bars and shower seats
- Special colors



026 ORANGE



035 RED



108 DARK BLUE



277 LIME GREEN

Handrails, shower head holder and shower curtain rails

- Standard colors



000 WHITE



112 ANTHRACITE

- Special colors



035 RED



108 DARK BLUE

Toilet seats and folding seats

- Standard colors



000 WHITE



112 ANTHRACITE

- Special colors



035 RED



108 DARK BLUE

NCS Color codes
000 = S0502-Y
026 = S1080-Y60R
035 = S4050-R
108 = S8010-R70B
112 = S6500-N
182 = S3000-N
277 = S1075-G50Y

Unlimited possibilities

Design and function go hand in hand in the wall track, which is the backbone of the flexible PLUS system.

Horizontal track makes it possible to move the products along the wall, while vertical track enables adjustment of their height. Changing the position of the products can be done without the use of tools. If the two tracks are combined, the available space in the bathroom can be utilized optimally, benefiting both user and caregiver.

If one or more products are not needed for a period of time, they can easily be removed from the track. The philosophy behind the flexible, intuitive bathroom is that the room should be tailored to the users and not the other way around.



Wall track



Horizontal wall track

R9824yyy Wall track, 94.5"

The track is available in various lengths.

Materials: Aluminium, plastic

Max. load: Dependent on wall construction - a vertical load up to 2200 lbs

yyy: Color of frontplate for track



Vertical wall track

R9864yyy Vertical wall track for height-adjustable products, 15.75"


Materials: Aluminium, plastic

Max. load: Dependent on wall construction - a vertical load up to 2200 lbs


yyy: Color of frontplate for track



A horizontal wall track allows for a flexible room layout. The track makes it possible to move the products sideways to suit spacial requirements and care needs.

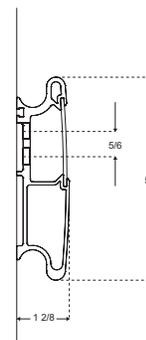

A vertical track makes it possible to adjust the products to a suitable height for the benefit of caregivers and patients alike.

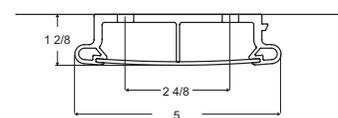

Sink brackets



⊕ Height-adjustable sink bracket

R4826 Manual sink bracket with crank handle

Suitable for MATRIX MEDIUM sink with integrated grab handles

Materials: Powder-coated steel and Polystyrene



⊕ Height-adjustable sink bracket

R4650 Air-assisted sink bracket with lever control

Suitable for MATRIX SMALL sink with integrated grab handles

Materials: Aluminium, plastic



⊕ Height-adjustable sink bracket

R4580yyy Manually operated sink bracket with lever control

Suitable for MATRIX SMALL sink with integrated grab handles

Materials: Aluminium, plastic

yyy: Color of frontplate for track



Fixed sink bracket


R2010 Fixed brackets for sinks, 2 pcs

Suitable for MATRIX MEDIUM, LARGE and ANGLE sinks with integrated grab handles.

Materials: Powder coated stainless steel

The MATRIX sink bracket is mounted directly on the wall, raised from the floor leaving room for the wheelchairs leg supports. With white installation cover including concealed flexible feed/waste system.

With the sink bracket mounted at the recommended height, the height of the sink can be adjusted 7.9" with an easy to reach crank handle.

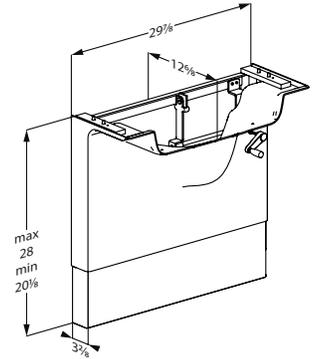


With the sink bracket mounted at the recommended height, the height of the sink can be adjusted by 11.75" from 26.25" to 38.2" above floor level.

R4650 is supplied with a air cylinder for sinks weighing 35 - 48 lbs as standard. However, it is possible to order R4650 with cylinders for sinks weighing 17 - 26 lbs and 26 - 35 lbs.



Its rounded, sleek design makes cleaning easy. The sink bracket suits most types of bolt-mounted sinks.

Height-adjustable sink brackets require a flexible feed and waste system, R2061. For further information see www.pressalitcare.com

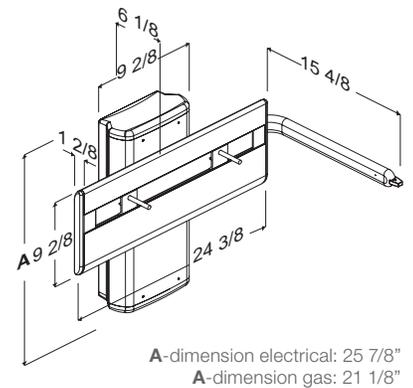

With the horizontal wall track mounted at the recommended height, the height of the sink can be adjusted by 9.5" from 2.75" to 36.25" above floor level.

Height-adjustable sink brackets require a flexible feed and waste system, R2061. For further information visit www.pressalitcare.com

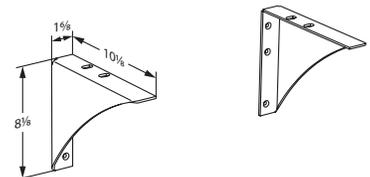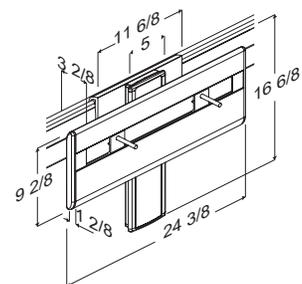

MATRIX Sinks



MATRIX SMALL sink

R2020000 Sink with grab bar handles and overflow

MATRIX Small is a narrow sink with two integrated grab bar handles in the front

Materials: Cultured marble with a high-polished, nonporous coating



MATRIX MEDIUM sink

R2120000 MATRIX MEDIUM vanity sink with integrated overflow and grab bar handles, 30.7" x 21.5"

LA1319 Drain fitting. Ring, chain and plug

LA1321 Drain fitting. Pop-up-plug

Materials: Cultured marble with a high-polished, nonporous coating

Suitable for mounting on MATRIX manual sink unit (R4826).



MATRIX LARGE sink

R2030000 Large Sink with storage surface, grab bar handles and overflow

MATRIX Large has two integrated grab bar handles on the front edge and on the sides

Materials: Cultured marble with a high-polished, nonporous coating



MATRIX ANGLE sink

R2040000 Angled Sink with storage surface, grab bar handles, overflow, right-facing

R2042000 Angled Sink with storage surface, grab bar handles, overflow, left-facing

MATRIX Angle has two integrated grab bar handles in the front

Materials: Cultured marble with a high-polished, nonporous coating



Faucet

RT622 Faucet, long rotatable spout


The ring-shaped operating lever is suitable for disabled users.
Ordered separately from sinks

Material: Chromium-plated brass

MATRIX Small has a flat bottom, and is wheelchair accessible. It is ideal for bathrooms where the available space has to be efficiently utilized .


The sink has raised edges, preventing water from running over the back or sides of the sink.

Faucet must be bought separately. Faucet hole: \varnothing 1.4"

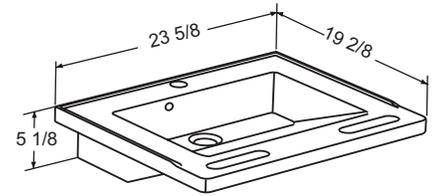

MATRIX MEDIUM vanity sink, 30.7" x 21.5" x 6.5", with two built in grab bar handles in the front, and integrated overflow. The sink has raised edges, preventing water from running over the back or down the outer sides.


Drain fitting and faucet must be bought separately. Faucet hole \varnothing 1.4"

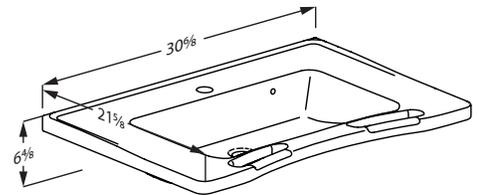

MATRIX LARGE sink is designed for a bathroom with plenty of space. The sink has a flat bottom, and is wheelchair accessible.

The sink has raised edges, preventing water from running over the back or sides of the sink.


Faucet must be bought separately. Faucet hole: \varnothing 1.4"

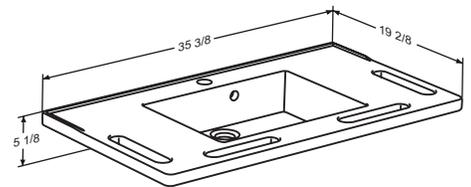


MATRIX ANGLE is designed so the sink can be positioned close to the toilet, while leaving a spacious work area for a helper. The sink is also ideal for mounting in a corner. The sink has a front drip tray to prevent water dripping onto the user. MATRIX Angle can only be mounted with either fixed brackets (R2010000) or manually adjustable brackets (R2015000).

The sinks has raised edges, preventing water from running over the back or sides of the sink.

Faucet must be bought separately. Faucet hole: \varnothing 1.4"

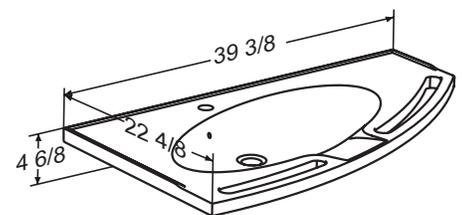

The long rotatable spout on RT622 makes it easier for seated users to reach the water and the spout can always be pushed to one side, if it is in the way. The symbols for hot and cold water are visible to both seated and standing users.

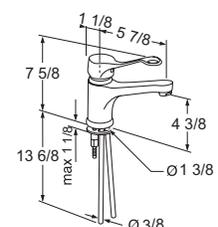

Fold-Down Grab Bars



⊕ Height-adjustable fold-down grab bars for wall track

R3585yyyxxx Fold-down grab bar, right-hand operated, length 33.5"

R3575yyyxxx Fold-down grab bar, left-hand operated, length 33.5"

R3560yyyxxx Fold-down grab bar, right-hand operated, length 23.6"

Materials: Aluminium, plastic, two-component plastic

Max. load: 330 lbs

yyy: Color of frontplate for track

xxx: Color of grab bar

⊕ Height-adjustable fold-down grab bars



R3185yyyxxx Fold-down grab bar, length 33.5"

R3160yyyxxx Fold-down grab bar, length 23.6"

Materials: Aluminium, plastic, two-component plastic

Max. load: 330 lbs

yyy: Color of frontplate for track

xxx: Color of fold-down grab bar

Fixed fold down grab bars



R3085xxx Fold down grab bar, length 33.5"

R3070xxx Fold down grab bar, length 27.6"

R3060xxx Fold down grab bar, length 23.6"

RF085xxx Fold down grab bar, length 33.5"

RF070xxx Fold down grab bar, length 27.6"




Paper holder

R9321 Toilet paper holder for fitting under the support arm

Ergonomically shaped fold-down grab bars provide a sense of security for seated users while also facilitating the task of transferring to and from wheelchairs. Fold-down grab bars mounted on both horizontal and vertical wall tracks ensure maximum flexibility in relation to individual user needs for space and support.

The end of the fold-down grab bar offers good support when sitting down or getting up. The non-slip surface ensures a firm yet comfortable grip.


Folding fold-down grab bars create more space in the room by folding away when not in use. With the horizontal wall track mounted at the recommended height, the height of the fold-down grab bar can be adjusted by 9.5" from 25.4" to 34.8" above the floor level.

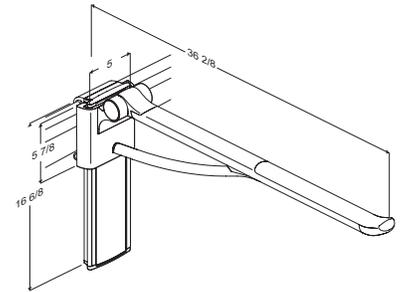

Ergonomically shaped fold-down grab bars provide a sense of security for seated users while also facilitating the task of transferring to and from wheelchairs. Height adjustment makes it possible to raise or lower the fold-down grab bars to provide the greatest possible support and comfort to the individual user.

The end of the fold-down grab bar offers good support when sitting down or getting up. The non-slip surface ensures a firm yet comfortable grip.


Folding fold-down grab bars create more space in the room by folding away when not in use. With the vertical wall track mounted at the recommended height, the height of the fold-down grab bar can be adjusted by 9.5" from 25.4" to 34.8" above the floor level.

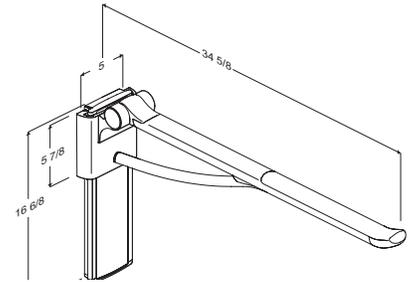


Ergonomically shaped fold-down grab bars provide a sense of security for seated users while also facilitating the task of transferring to and from wheelchairs.

The end of the fold-down grab bar offers good support when sitting down or getting up. The non-slip surface ensures a firm yet comfortable grip.

Folding fold-down grab bars create more space in the room by folding away when not in use.

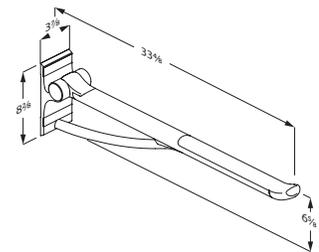

A paper holder mounted directly under the support arm ensures that the toilet paper is always within easy reach.

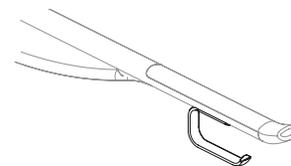

Shower Seats and Hand Rail



Height-adjustable shower seat 450

R7450yyyxxx Manually operated shower seat with backrest and armrest

Material: Aluminium, soft PUR foam with marine plywood core, plastic, two-component plastic

Max. load **Seat:** 330 lbs **Armrest:** 99 lbs

yyy: Color of vertical wall track

xxx: Color of shower seat, back- and armrest



Height-adjustable shower seat 310

R7320yyyxxx Manually adjusted shower seat with backrest and armrest

Material: Aluminium, soft PUR foam with marine plywood core, plastic, two-component plastic

Max. load **Seat:** 330 lbs **Armrest:** 99 lbs

yyy: Color of frontplate for track

xxx: Color of shower seat, back- and armrest



Fixed shower seat 450

R7408xxx Shower seat

Material: Aluminium, soft PUR foam with marine plywood core, plastic, two-component plastic

Max. load 330 lbs

xxx: Color of shower seat



Handrail combination

RT144xxx Handrail combination

Material: Powder-coated aluminium, plastic


Max. load 330 lbs

xxx: Color of handrail

With the horizontal wall track mounted at the recommended height, the height of the shower seat can be adjusted by 7.7" from 13.8" to 21.5" above floor level.

The shower seat is made from soft, non-slip materials which make bathing a pleasant and safe experience. Both the seat and the armrests can be folded up against the wall after use, creating space for standing users.


The seat, backrest and armrests can be adjusted simultaneously thanks to a safe and easy click system. The armrest can also be adjusted independently of the seat and backrest.

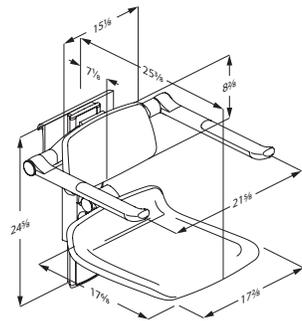

With the vertical wall track mounted at the recommended height, the height of the shower seat can be adjusted by 7.7" from 13.8" to 21.5" above floor level.


The shower seat is made from soft, non-slip materials which make bathing a pleasant and safe experience. Both the seat and the armrests can be folded up against the wall after use, creating space for standing users.

The seat, backrest and armrests can be adjusted simultaneously thanks to a safe and easy click system. The armrest can also be adjusted independently of the seat and backrest.

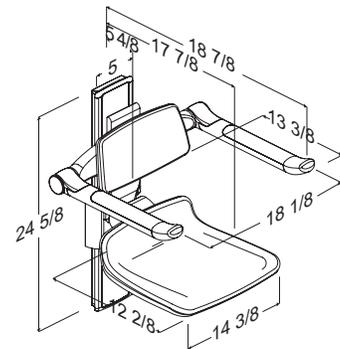


Mounted at the recommended height, the height of the shower seat will be 15.8" above floor level.

The shower seat is made from soft, non slip materials which make bathing a pleasant and safe experience. The seat can be folded up against the wall after use, creating space for standing users.

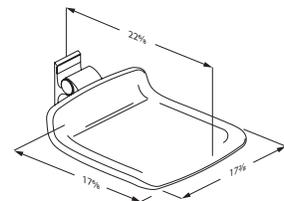

A handrail combination mounted by the shower provides both support, giving a sense of security and increased independence, as well as functions as a shower riser rail when mounting the shower head holder on the vertical handrail. Not ADA compliant

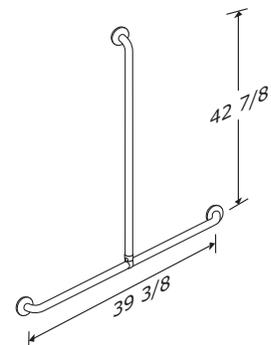

Adult Changing Tables



Height-Adjustable adult changing table 3000

- R8520021** Powered. Surface length 70.9", outer measurements: 38.9" x 75.2".
- R8516021** Powered. Surface length 51.2", outer measurements 38.9" x 55.5".
- R8516021** Powered. Surface length 70.9", outer measurements 40.4" x 75.2", incl. safety rail.
- R8516021** Powered. Surface length 51.2", outer measurements 40.4" x 55.9", incl. safety rail.



Fixed adult changing table 3000

- R8508021** Surface length 70.9", outer measurements 38.2" x 75.2".
- R8503021** Surface length 51.2", outer measurements 38.2" x 55.5".
- R8528021** Surface length 70.9", outer measurements 39.6" x 75.2", incl. safety rail.
- R8523021** Surface length 51.2", outer measurements 39.5" x 55.5", incl. safety rail.



Safety rail for adult changing table 3000

- R8511** Safety rail length 64.4" for retro-fitting on R8508021 and R8520021
- R8510** Safety rail length 47.2" for retro-fitting on R8503021 and R8516021



Height-adjustable adult changing table 2000

- ◀ **R8468000-93** Powered. Outer measurements: 34.7" x 70.9"
- R8478000-93** Powered. Outer measurements: 35.4" x 70.9", incl. safety rail.



Safety rail for adult changing table 2000


- RB2290000** Safety rail length 63", for retro-fitting on R8568000-93

Materials: Adult changing table 3000: Aluminum, Polyurethane foam, plastics, powder-coated steel. Adult changing table 2000: The frame is made from powder-coated stainless steel, the work surface is a mesh cover. The water collection tray is made from polystyrene.

When mounted at the recommended height, the height of the adult changing table can be adjusted from 12" to 40" above floor.

If a safety rail is to be pre-mounted, please be sure to indicate the correct item number when ordering.


Includes wired hand control and flexible hose, 60" and water collection tray with integrated outlet.

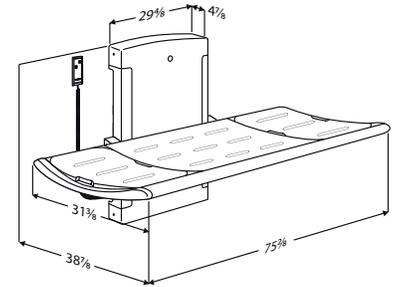

When mounted at the recommended height, the lying surface height is 34" above floor.


If a safety rail is to be pre-mounted, please be sure to indicate the correct item number when ordering.

Includes flexible hose, 60" and water collection tray with integrated outlet.

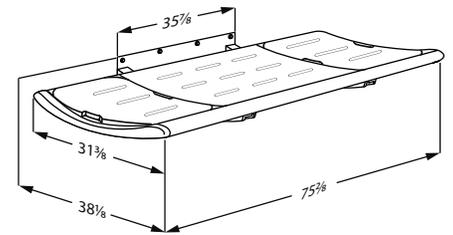

Provides safety for the user.


Can easily be folded up or down.

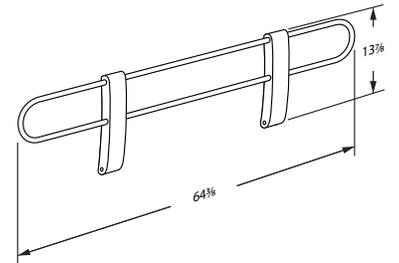

When mounted at the recommended height, the height of the adult changing table can be adjusted from 12" to 40" above floor.

If a safety rail is to be pre-mounted, please be sure to indicate the correct item number when ordering.

Includes wired hand control and flexible hose, 60" and water collection tray with integrated outlet.

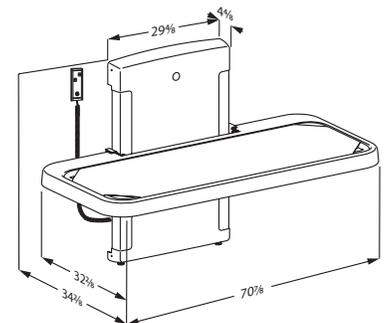

Enhances safety for the user.


Can easily be folded up or down.

Adult Changing Tables



Height-adjustable adult changing table 1000

- R8400000-93** Powered. Outer measurements: 28.9" x 55.1"
- R8401000-93** Powered. Outer measurements: 28.9" x 70.9"
- R8402000-93** Powered. Outer measurements: 30.1" x 55.1", incl. safety rail
- R8403000-93** Powered. Outer measurements: 30.1" x 70.9", incl. safety rail

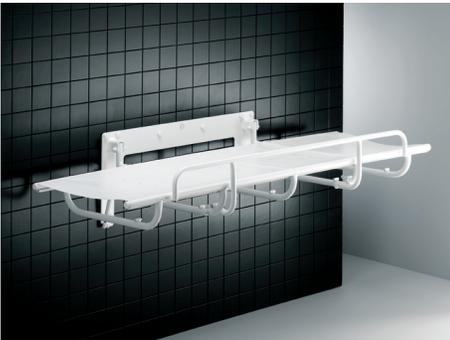

Fixed adult changing table 1000

- R8406000** Outer measurements 29.4" x 55.1", incl. safety rail
- R8407000** Outer measurements 29.4" x 70.9", incl. safety rail



Safety rail for adult changing table 1000

- R8410000** For retro-fitting on R8400000-93 and R8401000-93




Curtain for adult changing table 1000

- R8420000** For 55.1" table
- R8421000** For 70.9" table

When mounted at the recommended height, the height of the adult changing table can be adjusted from 12" to 40" above floor.


If a safety rail is to be pre-mounted, please be sure to indicate the correct item number when ordering.

Includes wired hand control and flexible hose, 60".

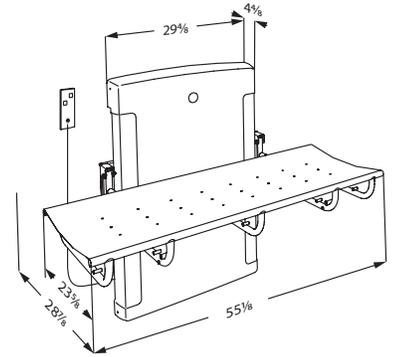

When mounted at the recommended height, the lying surface height is 34" above floor.


Includes flexible hose, 60".

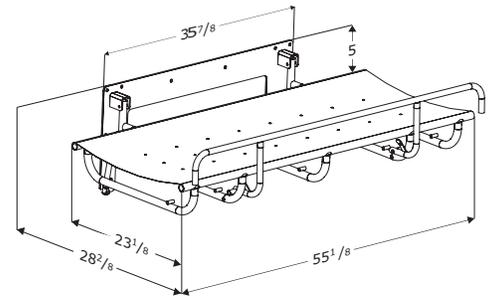

Enhances safety for the user.



A convenient splash curtain for the adult changing table, protects the caregiver against splashing water.

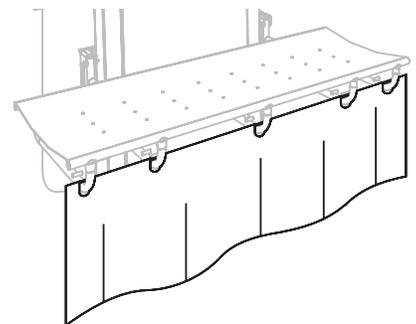

PRESSALIT® CARE

Reduced functionality need not limit a person's independence. For more than 35 years, the keep living philosophy has been a guiding principle for Pressalit Care's development of flexible kitchen and bathroom solutions for people with disabilities.

Thanks to its research-based knowledge, Pressalit Care is a market leader within specialist kitchen and bathroom solutions and today provides consultancy to customers worldwide.

An underlying respect for the individual and his or her possibilities is also reflected in the company's relationship to society at large and the local community. At Pressalit, Corporate Social Responsibility (CSR) is deeply rooted in the company's strategy and values and guides all parts of the organisation.

Pressalit Care's head office is located in Denmark.

www.pressalit.com

Pressalit A/S
Pressalitvej 1
8680 Ry
Denmark
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

Pressalit Inc. North America
P.O. Box 1624
East Dennis, MA 02641
USA
Tel.: 508 364 3804
E-mail: us@pressalit.com
Web: us.pressalit.com



Only for Adult
Changing Tables



All other products
in the leaflet

